CHLAMYDIA CEL

INSTRUCTIONS FOR USE

	English	2
	Français	3
	Deutsch	4
	Italiano	5
	Español	6
	Português	7
	Dansk	8
	Ελληνικά	9
	Magyar	10
	Diagram for Use	11
	Sensitivity, Specificity & Other Data	11
	Explanation of Symbols	12

EXPLANATION OF SYMBOLS

Consult Instructions for Use
 Consulter le Mode d'Emploi
 Gebrauchsanweisung
 Consultare le istruzioni per l'uso
 Consultar el manual de instrucciones
 Consultar o Folheto de Instruções
 Se brugsanvisningen
 Συμβουλευτείτε τις Οδηγίες Χρήσης
 Tanulmányozza a használati utasítást

In Vitro Diagnostic Medical Device
 Produit Diagnostique Médical In Vitro
 In-vitro-Diagnostika
 Diagnostico in vitro Dispositivo medico
 Producto Sanitario para Diagnóstico In Vitro
 Produto de Diagnóstico Médico In vitro
 Medicinsk udstyr til *in vitro*-diagnostik
 In Vitro Διαγνωστική Ιατρική Συσκευή
 In Vitro Orvosi Diagnosztikai Eszköz

Temperature Limitation
 Températures Limites
 Temperaturbegrenzung
 Temperatura limite
 Límite de temperatura
 Temperatura Limitada
 Opbevarés ved
 Περιορισμός Θερμοκρασίας
 Tárolási hőmérséklet

Biological Risks
 Risques Biologiques
 Biogefährdung
 Rischi biologici
 Riesgo biológico
 Riscos Biológicos
 Biologisk risiko
 Βιολογικοί Κίνδυνοι
 Biológiai veszély

Authorised Representative in the EC
 Représentant Autorisé dans la EC
 Bevollmächtigter in der Europäischen Gemeinschaft
 Rappresentante Autorizzato nella EC
 Representante autorizado en la EC
 Representante Autorizado na CE
 Repräsentant
 Εξουσιοδοτημένος Αντιπρόσωπος στην EC
 Felhatalmazott képviselő az EU-ban

MDCI Ltd
 Arundel House, 1 Liverpool Gardens
 Worthing, West Sussex BN11 1SL
 United Kingdom

Manufacturer
 Fabricant
 Hersteller
 Fabbricante
 Fabricante
 Fabricante
 Fabricante
 Fabrikant
 Κατασκευαστής
 Gyártó

Cellabs Pty Ltd
 Unit 7, 27 Dale Street (PO Box 421)
 Brookvale, NSW 2100 Australia
 Tel: +61 2 9905 0133 Fax: +61 2 9905 6426
 Web: <http://www.cellabs.com.au>
 Email: sales@cellabs.com.au

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LC1.09 - PDF
 28th November 2006

CHLAMYDIA CEL

INTENDED USE AND PRINCIPLE OF THE TEST

The Chlamydia Cel IF Test is a rapid *in vitro* direct immunofluorescence test for the detection and diagnosis of *Chlamydia trachomatis* organisms in patient specimens. The fluorescein-labelled mouse monoclonal antibody reagent binds specifically to *C. trachomatis* in the specimen. *C. trachomatis* display bright green fluorescence with typical morphology.

CONTENTS OF THE KIT

		KC1 Demo	KC1 Standard	Bulk
RC1	Chlamydia Cel Reagent	0.25mL	1.25mL	5mL
C	Positive Control Slide (Single Use only)	1	1	-
RMF	Mounting Fluid	2.5mL	2.5mL	-
	<i>Tests</i>	<i>10</i>	<i>50</i>	<i>200</i>

Materials are supplied ready for use. Store at 2-8°C. Expiry dates are clearly marked on each kit component and on the box. Expiry dates do not change once opened.

MATERIALS REQUIRED BUT NOT PROVIDED

Microscope slides with 6-8mm diameter wells; precision pipette for delivering 25µL; methanol for specimen fixation; humid chamber; wash bath; phosphate buffered saline (PBS) for washing step; cover slips; non-fluorescing immersion oil; and fluorescence microscope with filter system for FITC (maximum excitation wavelength 490nm, mean emission wavelength 530nm) and x600-x1000 magnification.

PRECAUTIONS

For *in vitro* diagnostic use only. Do not use after the expiry date shown on the label. If protective packaging is damaged, contact your local distributor and ask for a replacement. Do not mix components from different kits. The Chlamydia Cel Reagent has been optimised for use with Cellabs Positive Control Slide and Mounting Fluid. Evans Blue dye contained in the test reagent is a possible carcinogen, therefore avoid contact with the skin. Patient specimens and the positive control slide should be handled as though potentially infectious. A Positive Control Slide must be run with each test run. Refer to Material Safety Data Sheet (MSDS) for further information.

SPECIMEN COLLECTION

Cervical Specimens: Specimens from the female cervix should contain as many columnar epithelial cells as possible as *C. trachomatis* are intracellular organisms that infect these cells. Clean cervix with a sterile cotton swab before sampling. Insert swab in the cervical canal and rotate swab at the squamo-columnar junction. Withdraw swab without touching any vaginal surfaces.

Urethral Specimens: Specimens from the male urethra should also contain intact epithelial cells to ensure an accurate diagnosis. Patients preferably should not have urinated one hour before collection. Insert swab 2-4cm into the urethra using dacron or cotton wool-tipped thin swabs. Rotate the swab and withdraw.

Conjunctival Specimens: Using a sterile cotton-tipped swab, vigorously scrape the lower lid conjunctivae of affected eyes to obtain a good yield of epithelial cells. If both eyes are affected, first swab the least affected eye then swab the most affected eye.

PREPARATION OF SLIDES

Roll the swab back and forth on the glass slide to cover the area of the well. Allow the specimen to air dry, then fix in methanol for 5 minutes. Drain and air dry. If the specimen is not tested immediately, store at 2-8°C overnight or freeze at -20°C for up to 2 months.

INSTRUCTIONS FOR USE

- Add 25µL of **RC1** to the fixed specimen and **C**, covering well area.
- Incubate the slides at 37°C in a humid chamber for 30 minutes. Do not allow the slides to dry as this may cause non-specific binding.
- Rinse gently in a bath of PBS for one minute.
- Drain slide and remove excess moisture around well with tissue.
- Add a drop of **RMF** to the slide well. Place a coverslip on top of the drop and remove air bubbles.
- Scan the entire specimen using a fluorescence microscope under oil immersion at x600-x1000 magnification. Read immediately or store at 2-8°C in the dark for up to 24 hours.

READING AND INTERPRETATION OF RESULTS AND DIAGNOSIS

Extracellular elementary bodies (EB's) appear as bright apple-green fluorescent pin-point, smooth-edged disc shaped bodies (approximately 300nm in diameter) and can be seen against a background of reddish-brown counterstained cells. Reticulate bodies may also be observed and are 2-3 times larger than the EB's and they fluoresce evenly or possess dark centres with a halo of fluorescence. Intact intracellular chlamydial inclusions are rarely seen. The control slide should be used for comparison with the appearance and size of EB's found in the specimen. A positive diagnosis is made when fixed stained specimens show at least 10 chlamydial bodies. A negative diagnosis is reported when fixed stain smears are free of chlamydial organisms but at least 10 columnar cells (intact or ruptured) are present. Irregularly shaped fluorescent material that differs in size from chlamydial bodies described above or fluoresces white, red or yellow should be considered non-specific staining.

WASTE DISPOSAL

Dispose of any unused components as biohazardous waste. Where the test reagent has been disposed of in the sink, ensure it is flushed with large quantities of water (as the sodium azide it contains may react with copper/lead plumbing systems). For more information, please refer to the MSDS.

SENSITIVITY, SPECIFICITY, & OTHER DATA ON CHLAMYDIA CEL

Refer to summary table at end of insert. All data on the Chlamydia Cel can be obtained in the product information sheet. Please ask your local distributor or contact Cellabs.

INDEMNITY NOTICE

Modifications or changes made in the recommended procedure may affect the stated or implied claims. A positive or negative result does not preclude the presence of other underlying causative agents. Cellabs and its agents and distributors shall not be liable for damages under these circumstances.

FIGURE 1: CHLAMYDIA CEL DIAGRAM FOR USE

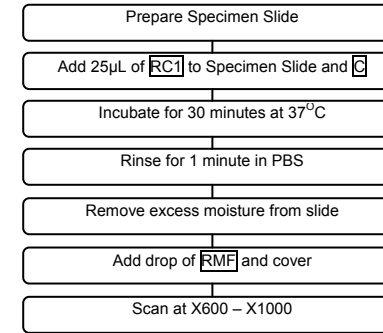
TABLE 1: SENSITIVITY, SPECIFICITY, & OTHER DATA ON THE CHLAMYDIA CEL
 TABLEAU 1: SENSIBILITÉ, SPÉCIFICITÉ ET AUTRES DONNÉES DU TEST CHLAMYDIA CEL
 TABELLE 1: SENSITIVITÄT, SPEZIFITÄT UND ANDERE DATEN ZUM CHLAMYDIA CEL
 TABELLA 1: SENSIBILITA' SPECIFICITA' ED ALTRI DATI SULLA CHLAMYDIA CEL
 TABLA 1: SENSIBILIDAD, ESPECIFICIDAD Y OTROS DATOS DEL CHLAMYDIA CEL
 TABELA 1: SENSIBILIDADE, ESPECIFICIDADE, & OUTROS DADOS DO CHLAMYDIA CEL
 TABEL 1: SENSITIVITET, SPECIFICITET OG ANDRE OM OPLYSNINGER OM CHLAMYDIA CEL
 ΠΙΝΑΚΑΣ 1: ΕΥΑΙΣΘΗΣΙΑ, ΕΙΔΙΚΟΤΗΤΑ & ΆΛΛΑ ΔΕΔΟΜΕΝΑ ΤΗΣ ΕΞΕΤΑΣΗΣ CHLAMYDIA CEL
 1. TÁBLÁZAT: A CHLAMYDIA CEL ÉRZÉKENYSÉGI, SPECIFICITÁSI ÉS EGYÉB ADATAI

Trial	Sensitivity	Specificity	Repeatability	Reproducibility
Essai	Sensibilité	Spécificité	Répétabilité	Reproductibilité
Versuch	Sensitivität	Spezifität	Wiederholpräzision	Reproduzierbarkeit
Prova	Sensibilita'	Specificita'	Ripetibilità	Riproducibilità
Prueba	Sensibilidad	Especificidad	Repetibilidad	Reproducibilidad
Teste	Sensibilidade	Especificidade	Repetição	Reprodutibilidade
Forsøg	Sensitivitet	Specificitet	Repeterbathed	Reproducerbarhed
Δοκιμασία	Ευαισθησία	Ειδικότητα	Επαναληψιμότητα	Αναπαραγωγιμότητα
Vizsgálat	Érzékenység	Specifitás	Ismételhetőség	Reprodukálhatóság
A	100%	99.4%	-	-
B	95.8%	100%	-	-
C	100%	100%	-	-
D	100%	100%	-	-
E	100%	99.5%	-	-
F	100%	100%	-	-
G	100%	100%	-	-
H	-	-	100% Correlation	100% Correlation
	Not cross reactive with / Pas de Réaction Croisée avec / Keine Kreuzreaktionen mit / Non mostra reazione crociata con / No muestra reacción cruzada con / Não apresenta reacções cruzadas com / Krydsreagerer ikke med / Δεν υπάρχει διασταυρωμένη αντιδραστικότητα με / Nem keresztreagál:			
	Beta haemolytic Streptococcus group B, Beta haemolytic Streptococcus group A, Lactobacillus acidophilus, Peptostreptococcus anaerobius, Proteus mirabilis, Escherichia coli, Streptococcus faecalis, Staphylococcus aureus, Pseudomonas aeruginosa, Staphylococcus epidermidis, Bacteriodes melaninogenicus, Actinomyces israeli, Gardnerella vaginalis, Neisseria gonorrhoeae, Candida albicans, Trichomonas vaginalis;			

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